

PRESTIGE HYPERBARIC

Elevate Your Recovery

Grand Vessel of Power

Grand Vessel of Power — 2.0 ATA Hardshell Lay-Down — Owner's Manual & Product Guide

The Grand Vessel of Power is a large hard-shell lay-down hyperbaric chamber from Prestige Hyperbaric. Engineered with precision, the Grand Vessel delivers oxygen therapy at pressures up to 2.0 ATA with fast pressurization and a clinical-grade build — the gold standard for serious hyperbaric therapy.

Key Features

- **Transparent Windows:** Aviation-grade, double-layer polycarbonate windows reduce claustrophobia and brighten the space.
- **Advanced Construction:** Precision-welded frame with multi-thickness steel reinforcement for high-pressure stability.
- **Dual Controls:** Inside + outside control panels and dual screens for temperature, humidity, and O2 purity.
- **Power-Failure Safety:** Automatic, gradual depressurization and safe door release.
- **Smart Door Sensor:** Operates only when fully closed; indicator lights inside and out.
- **Adjustable Working Pressure:** User-settable; follow your clinician's guidance for protocols.
- **Efficient Air Cooling:** Electric, maintenance-free cooling and dehumidification (no ice water/Freon).
- **Clean-Air Suite:** Anion PM2.5 purification + nano-water sterilization system.

Safety Highlights

- **WARNING — Do Not Sleep Inside the Chamber:** Never fall asleep or remain unattended while inside the hyperbaric chamber. Sessions must be conducted while the user is awake and alert at all times. In the event of discomfort, dizziness, or any unusual sensation, immediately signal an attendant or activate the pressure-release mechanism to exit safely.
- Operate only under appropriate guidance and according to the protocol recommended by a qualified professional.
- Do not use the chamber if any zipper, valve, gauge, hose, regulator, concentrator, or relief valve appears damaged.
- Keep the chamber away from heat sources, sparks, open flame, smoking materials, oils, and flammable products.
- Maintain clear communication with an attendant during use whenever possible.
- Use in a clean, dry, climate-controlled environment.
- Dual digital displays (where equipped) show O2 purity, temperature, and humidity.
- Door safety sensor prevents improper operation (hard-shell units).
- Redundant relief valves: primary + backup + emergency.
- HD phone intercom for clear communication (where equipped).
- Safe exit even during power outage after controlled depressurization.

Technical Specifications

Feature	Details
---------	---------

Model	Grand Vessel of Power
Manufacturer	Prestige Hyperbaric
Shell Type	Hard-Shell Lay-Down
Capacity	1 person (very spacious; may accommodate 2)
Max Pressure	Up to 2.0 ATA
Oxygen Flow	10 L/min standard; 20 L/min with additional concentrator
Cooling & Dehumidification	Electric air-cooling; maintenance-free; no ice water or Freon
Purification	Anion PM2.5 purification + nano-water sterilization
Controls & Readouts	Internal & external control panels; dual digital screens
Retail Price	\$33,000

Delivery & Installation Options

- **Free Curbside Delivery** — Carrier drops the crated unit at your curb. You handle offload, uncrate, move, and placement.
- **Professional Install** — We set up, calibrate, and test. Customer is responsible for moving equipment into the install room.
- **White Glove Delivery** — We handle everything: offload, uncrate, placement, on-site education, and cleanup.

Warranty

3-Year Residential Warranty / 1-Year Commercial Warranty

- **3-Year Residential Warranty (Personal Use):** Prestige Hyperbaric equipment includes a 3-year warranty when used in a personal residential setting, covering defects in materials and workmanship.
- **1-Year Commercial Warranty (Commercial / Clinical Use):** Units used in wellness centers, hyperbaric clinics, medical offices, spas, gyms, recovery studios, or any facility providing paid sessions to third parties are covered by a 1-year warranty.
- **Extended Coverage Available:** Extended warranty coverage is available for \$2,000 per additional year.
- Coverage may be refused in cases of misuse, unauthorized modification, non-professional installation where required, improper electrical setup, or use in non-climate-controlled environments. For a no-hassle experience, purchase professional installation or White Glove Delivery.

Returns

- Full refund only if canceled within 24 hours of ordering.
- After 24 hours or once shipped, orders are treated as custom and final sale unless an exception is approved.
- If a shipped return is approved, a 40% restocking fee applies and all return shipping and related costs are deducted from the refund.

Support & Contact

Website: www.PrestigeHyperbaric.com

Phone/Text: 727-591-4268

Email: prestigehyperbaric@gmail.com

Tagline: Elevate Your Recovery