

PRESTIGE HYPERBARIC

Elevate Your Recovery

Sphere of Power

Sphere of Power — 1.5 ATA Softshell Sitting — Owner's Manual & Product Guide

The Sphere of Power is a soft-shell sit-up hyperbaric capsule for users who prefer an upright position. It is portable, easy to install, and designed for accessible home or clinic use at up to 1.5 ATA.

Key Features

- Sit-up capsule format for upright positioning.
- Compact footprint for home wellness spaces.
- Soft-shell chamber design — lightweight, portable, and easier to place than hard-shell systems.
- Inflatable pre-pressurization design for a comfortable, accessible treatment experience.
- Clear vinyl window for visibility and reduced claustrophobia.
- External valve and pressure regulator for simple, reliable pressure control.
- Electric, maintenance-free air cooling and dehumidification (no ice water or Freon required).
- Anion PM2.5 air purification + nano-water sterilization system.
- 10 L/min oxygen flow standard; 20 L/min with additional concentrator.
- Redundant safety relief valves — primary, backup, and emergency.
- Automatic safe depressurization on power failure.

Safety Highlights

- **WARNING — Do Not Sleep Inside the Chamber:** Never fall asleep or remain unattended while inside the hyperbaric chamber. Sessions must be conducted while the user is awake and alert at all times. In the event of discomfort, dizziness, or any unusual sensation, immediately signal an attendant or activate the pressure-release mechanism to exit safely.
- Operate only under appropriate guidance and according to the protocol recommended by a qualified professional.
- Do not use the chamber if any zipper, valve, gauge, hose, regulator, concentrator, or relief valve appears damaged.
- Keep the chamber away from heat sources, sparks, open flame, smoking materials, oils, and flammable products.
- Maintain clear communication with an attendant during use whenever possible.
- Use in a clean, dry, climate-controlled environment.
- Dual digital displays (where equipped) show O₂ purity, temperature, and humidity.
- Door safety sensor prevents improper operation (hard-shell units).
- Redundant relief valves: primary + backup + emergency.
- HD phone intercom for clear communication (where equipped).
- Safe exit even during power outage after controlled depressurization.

Technical Specifications

Feature	Details
---------	---------

Model	Sphere of Power
Manufacturer	Prestige Hyperbaric
Shell Type	Soft-Shell Sit-Up
Capacity	1 person (spacious)
Max Pressure	Up to 1.5 ATA
Windows	Clear vinyl window
Controls	External valve & regulator
Oxygen Flow	10 L/min standard; 20 L/min with additional concentrator
Cooling	Electric, maintenance-free air cooling and dehumidification
Air Purification	Anion PM2.5 + nano-water sterilization system
Safety Valves	Primary, backup & emergency
Power Failure	Automatic safe depressurization
Retail Price	\$10,999

Delivery & Installation Options

- **Free Curbside Delivery** — Carrier drops the crated unit at your curb. You handle offload, uncrate, move, and placement.
- **Professional Install** — We set up, calibrate, and test. Customer is responsible for moving equipment into the install room.
- **White Glove Delivery** — We handle everything: offload, uncrate, placement, on-site education, and cleanup.

Warranty

3-Year Residential Warranty / 1-Year Commercial Warranty

- **3-Year Residential Warranty (Personal Use):** Prestige Hyperbaric equipment includes a 3-year warranty when used in a personal residential setting, covering defects in materials and workmanship.
- **1-Year Commercial Warranty (Commercial / Clinical Use):** Units used in wellness centers, hyperbaric clinics, medical offices, spas, gyms, recovery studios, or any facility providing paid sessions to third parties are covered by a 1-year warranty.
- **Extended Coverage Available:** Extended warranty coverage is available for \$2,000 per additional year.
- Coverage may be refused in cases of misuse, unauthorized modification, non-professional installation where required, improper electrical setup, or use in non-climate-controlled environments. For a no-hassle experience, purchase professional installation or White Glove Delivery.

Returns

- Full refund only if canceled within 24 hours of ordering.

- After 24 hours or once shipped, orders are treated as custom and final sale unless an exception is approved.
- If a shipped return is approved, a 40% restocking fee applies and all return shipping and related costs are deducted from the refund.

Support & Contact

Website: www.PrestigeHyperbaric.com

Phone/Text: 727-591-4268

Email: prestigehyperbaric@gmail.com

Tagline: Elevate Your Recovery